

INTENDED PURPOSE

Allows self-collection of a capillary blood sample in conjunction with a compatible sample collection reservoir that is shipped to an accredited laboratory for analysis.

KIT CONTENTS

- Tasso Blood Lancing Device
- Compatible Tube, refer to manufacturer's labeling
- Heat Pack
- Bandage
- Alcohol Swab
- Return Bag with Absorbent Pad
- Instructions for Use

DISPOSAL

Follow local guidelines for discarding used or defective sharps as a biohazard material.

WARNINGS

- Prescription use only.
- Do not use after expiration date.
- Single use only.
- Keep out of reach of children.
- Minor bruising or residual marks may occur at the sample collection site.
- Do not resterilize.
- Do not use if device packaging has been opened or damaged.
- For external use only.
- Intended for self-use or use by a healthcare provider.
- Use while seated as fainting may occur with any blood sampling procedure.
- Refer to manufacturer's instructions for individual components.
- Not for use on infant heels.
- Users must validate use of product for their specific diagnostic assay.

TRANSLATED INSTRUCTIONS

- The Tasso+ Kit instructions for use and translations can be found on the Tasso website:
- Die Gebrauchsanweisung für das Tasso+ Kit und die Übersetzungen finden Sie auf der Tasso-Website:
- Las instrucciones de uso y las traducciones del kit Tasso+ se pueden encontrar en el sitio web de Tasso:
- Le istruzioni per l'uso del kit Tasso+ e le relative traduzioni sono reperibili sul sito Web Tasso:
- Les instructions d'utilisation d'origines du kit Tasso+ ainsi que les traductions correspondantes sont disponibles sur le site web de Tasso:
- De gebruiksaanwijzing voor de Tasso+ Kit en vertalingen zijn beschikbaar op de Tasso website:
- Bruksanvisningen och översättningarna till Tasso+ Kit finns på Tassos webbplats:
- A Tasso+ Kit használati utasítása és annak fordításai megtalálhatók a Tasso weboldalán:
- Návod na použitie a preklady pre sadu Tasso+ nájdete na webovej stránke spoločnosti Tasso:
- Tasso+ -sættets brugsanvisning og oversættelser kan findes på Tassos hjemmeside:
- Bruksanvisninger for Tasso+ Kit og oversættelser findes på Tassos nettsted:
- Tasso+ -setin käyttöohjeet ja käännökset löytyvät Tasson verkkosivuilta:

<https://www.tassoinc.com/eu/tassoplus-kit-ifu>

Any serious incident that has occurred in relation to the device should be reported to the manufacturer, the authorized representative, and the competent authority of the Member State in which the user and/or patient is established.

SYMBOLS


REF Catalog number

LOT Lot number

 Use by date


 Single use only

 Store at 18-25°C


 Transport at 25-85% RH

 2797 CE Mark

MD Medical Device

 Consult instructions for use

 This device contains sharps

 Do not use if package is damaged

Rx ONLY Prescription use only

STERILE R Sterilized by irradiation

 Do not resterilize

 Importer

 Tasso, Inc.
400 2nd Ave. W
Seattle, WA 98119 USA
www.tassoinc.com


EC REP Emergo Europe BV
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

 MedEnvoy
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands

PN0682-01, 03

 Tasso

Tasso+ Kit
Instructions For Use

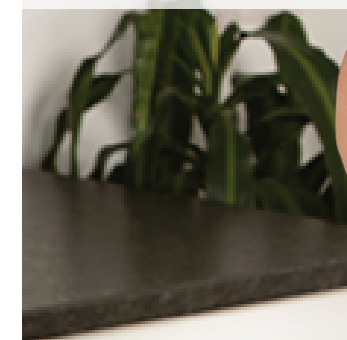

Additional Supplies:

Timer 


Pen 

Razor 


1. Prepare



1. Wash hands before use. Healthcare workers must wear gloves during collection.

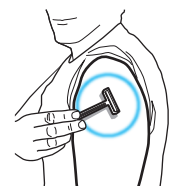

2. Device will be placed on upper arm - for best results remove any excess hair.

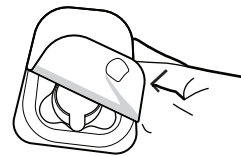


3. Collect all needed items. Remove button from tray.



4. Twist and remove cap from tube.




5. Press tube into device until snug with fill lines facing out.

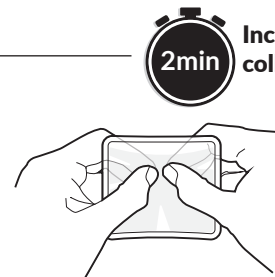


2min Increase blood flow at collection site

6. Follow heat pack instructions to activate. If pack does not heat up, discard and use hand instead.

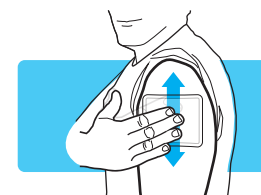

7. Rub heat pack or hand up and down user's shoulder for 2 minutes to warm and increase blood flow.

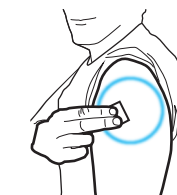

8. Clean area with alcohol pad and allow to dry.




9. Remove clear cover from red button.

2. Collect




You may not feel the device. Leave on 5 minutes OR less if blood is at fill line.

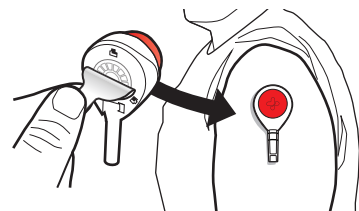


10. Peel tab away from device and stick device to upper arm. Hang arm straight down at side.



11. Press button all the way down ONCE and release. Set timer for 5 minutes. Watch tube fill using mirror as needed.

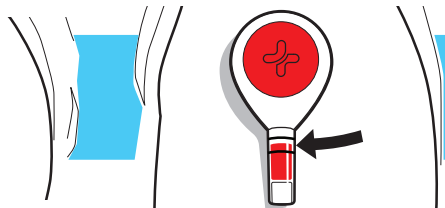

12. Blood may not appear for first 1-2 minutes. Remove device at 5 total minutes or sooner if blood reaches top line before time is up. Do not fill past top line!

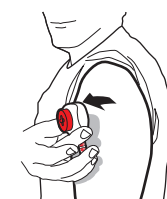

13. Remove device by peeling from one side. If needed, apply supplied bandage.



14. Remove tube from device with a slight twist and pull down.



15. Snap cap fully onto tube.



16. Gently invert tube 10 times to mix the blood. Tap or flick tube to get blood moving. Any gel in tube bottom will not mix with blood.


3. Ship




17. Write user's Date of Birth (MM/DD/YYYY) on tube label in the space above the barcode.



18. Place tube in clear return bag and press to seal. Do not remove pad.

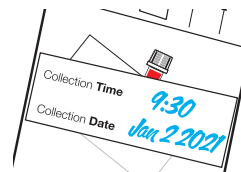

19. Record time and date on return bag label.

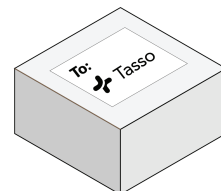

20. If returning blood sample to a lab via parcel service, follow provided shipping instructions.



21. Follow local guidelines for discarding used materials.



22. Wash hands after use.



First time using Tasso? Scan the QR code to access our instructional video or enter the URL below in your web browser:

www.tassoinc.com/tassoplus-video

Contact us for help: +1 800-257-2370