

PACKAGE INSERT INSTRUCTIONS FOR USE



1. Wash hands before use.
Healthcare workers must wear gloves during collection.




2. Remove button from pouch.



3. Twist off cap from compatible tube.



4. With fill lines facing out, press tube into device until snug.

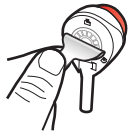


5. Peel tab away from red button.



6. Remove clear cover from red button.



7. Prepare shoulder. Stick device to clean shoulder and press button to start collection.

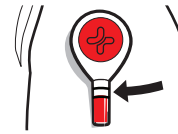

8. Allow tube to fill for 5 minutes. Remove device if blood reaches top fill line before time is up.



9. Remove device by peeling from one side.



10. Remove tube from button with a slight twist and pull down.



11. Snap cap fully onto the tube. Mix per manufacturer's labeling.

**Contact us
for help:
1-800-257-2370**

TRANSLATED INSTRUCTIONS

- The TassoOne Plus instructions for use and translations can be found on the Tasso website:
- Die Gebrauchsanweisung und die Übersetzungen von TassoOne Plus finden Sie auf der Tasso-Website:
- Las instrucciones de uso de TassoOne Plus y las traducciones se pueden encontrar en el sitio web de Tasso:
- Le istruzioni per l'uso e le traduzioni di TassoOne Plus sono disponibili sul sito Web di Tasso:
- La notice d'utilisation du TassoOne Plus et des versions traduites sont disponibles sur le site Web de Tasso :
- De gebruiksaanwijzing voor de TassoOne Plus en vertalingen zijn te vinden op de Tasso-website:

- Instruktionerna för användning och översättningar tillhörande TassoOne Plus finns på Tassos webbplats:
- A TassoOne Plus használati utasítása és annak fordításai megtalálhatók a Tasso weboldalon:
- Návod na použitie a preklady TassoOne Plus nájdete na webovej stránke spoločnosti Tasso:
- TassoOne Plus-brugsanvisninger og -oversættelser findes på Tassos websted:
- TassoOne Plus-bruksanvisningen og -oversættelser finnes på Tasso-nettstedet:
- TassoOne Plus -käyttöohjeet ja käänökset löytyvät Tasson verkkosivuilta:

<https://www.tassoinc.com/eu/tassoone-plus-ifu>

INTENDED PURPOSE

Allows self-collection of a capillary blood sample in conjunction with a compatible sample collection reservoir.

MATERIALS PROVIDED

Each box contains one TassoOne Plus device.

DISPOSAL

Follow local guidelines for discarding used or defective sharps as a biohazard material.

MATERIALS REQUIRED (Not provided)



- Compatible tube, refer to manufacturer's labeling
- Alcohol swab
- Gloves (for Healthcare Providers)
- Bandage
- Heat pack or other arm warming method

WARNINGS

- Prescription use only.
- Single use only.
- Do not resterilize.
- Do not use if pouch has been opened or damaged.
- Keep out of reach of children.
- Minor bruising or residual marks may occur at the sample collection site.
- Do not use after expiration date.
- Intended for self-use or use by a healthcare provider.
- Use while seated as fainting may occur with any blood sampling procedure.
- Not for use on infant heels.
- Users must validate use of product for their specific diagnostic assay.
- For external use only.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer, the authorized representative, and the competent authority of the Member State in which the user and/or patient is established.

SYMBOLS

- | | | | |
|---|------------------------|---|----------------------------------|
|  | Catalog number |  | Consult instructions for use |
|  | Lot number |  | This device contains sharps |
|  | Use by date |  | Do not use if package is damaged |
|  | Single use only |  | Prescription use only |
|  | Store at 18-28°C |  | Sterilized by irradiation |
|  | Transport at 25-85% RH |  | Do not resterilize |

 CE Mark

 Importer

 Medical Device